


TECNICAL SHEET

# IKI QUIRURGIC IIR

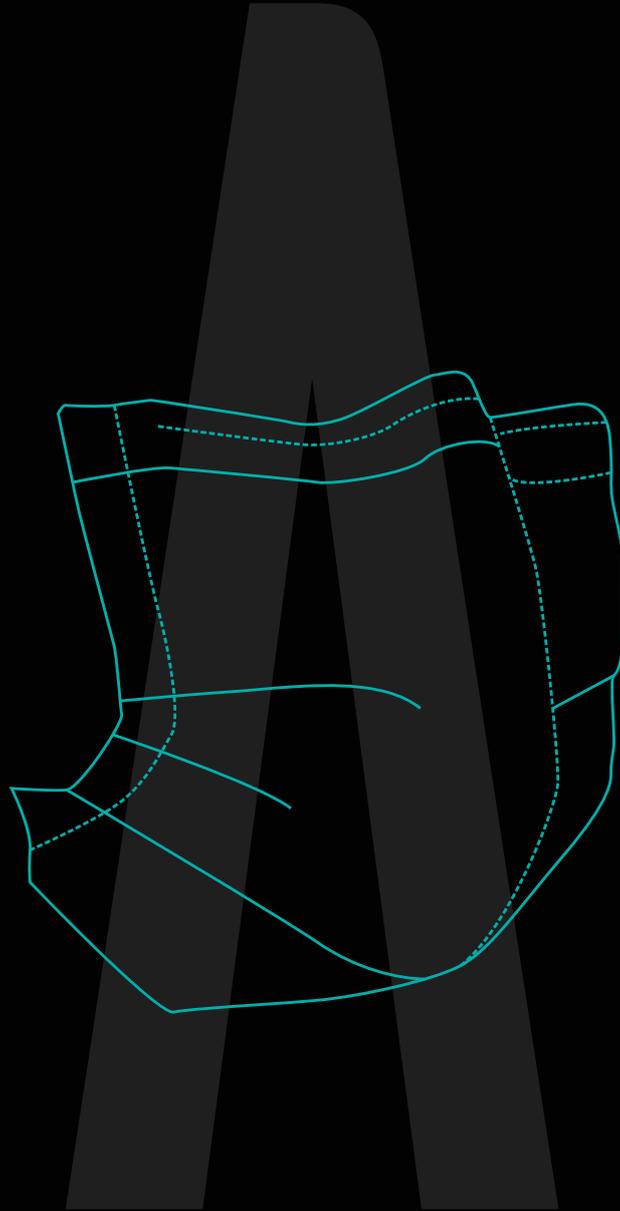

# TECNICAL SHEET

## DESCRIPTION:

Surgical face mask available in 3 sizes and 3 colors, perfectly adjusting to our anthropometric profile according to the face size. With an elastic band that is much more comfortable to prolonge the usage and better feel on the face. All the masks are packed with Ozone during their packaging process, so the masks are disinfected and free from contact with the outside while maintaining their barrier film, which will prevent the appearance of reactions or other infections derived from the continued use of the mask. The materials used on the making process are 100% recyclable and 100% free of polyurethanes.

## CARACTERÍSTICAS:

MODEL	SIZE	COLOR
XS	140x95	  
S	160x95	  
M	180x95	  

the measurements (10mm-15mm) allows a full airway coverage with full extension dimensions for each profile. fil.

## TECNIC SPECIFICATIONS:

Contribution with Ozone (O3), Concentrated in 70g N/m<sup>3</sup> before it's sealing packaging, gives a total disinfection tested by laboratories. Surgical Mask - Tipe IIR formed by 3 layers according to UNE-EN 14683:2019+AC2019 with the regulation measurements according to UNE-EN 0064-1:2020.

- Layer TNT 1 Spunbond color 25gr/m<sup>2</sup>
- Layer TNT 2 Meltblown 25gr/m<sup>2</sup>
- Layer TNT 3 Spunbond White 25gr/m<sup>2</sup>
- Elastic band 12mm ergonomic and 100% recycled with elongation factor of >150%

BFE > 99%, with laboratory results not less than a 99,98 Splash resistance >16 KPa Breathability < 60 Pa/cm<sup>2</sup>

Microbial cleanliness: ≤30 UFC/g

## ADDITIONAL INFORMATION :

Protection of health personnel and patients from airborne infectious diseases as long as they are not susceptible or are not recommended to use surgical masks or filters to protect against particles. In case of doubt or problems arising from its use, consult your doctor.

- Packaged in a sealed film with 10 units.
- Protect from light sources and protect from moisture.
- Single-use “non-reusable” product. Do not use the product on more than one user or multiple times on the same user.
- Expiration of two years from date of manufacture.
- Deposit the used material in packaging and in the organic waste deposit.

# METHOD OF USE

For its correct placement, the outer layer must be identified, being this color according to the model (blue, pink or black).

Since the masks used are considered highly contaminated, it is essential that:

- The body of the mask is not touched by the fingers/hands of the wearer;
- The hands are disinfected (total disinfection of the same) after the removal of the mask;
- Place the mask that covers the user's nose and mouth, never let a mask hang from the user's neck.
- A used mask should be discarded when it is no longer needed or between two procedures; when there is an additional need for protection, the user should put on a new mask.



Before putting on or removing the mask, you must first wash your hands with soap and water, or a hydroalcoholic solution.



Put on the mask, adjust the clip on the nose and cover the mouth.



Adjust the elastic behind the ears.




Do not wear the mask for more than 4 hours or touch it with your hands during use.



Replace with another if it gets wet or damaged.



Remove it without touching the front part.

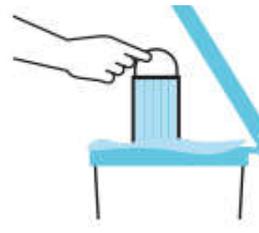

Discard after use in a container provided with a plastic bag.



During or after use it should not be placed on the forehead or chin.

# COLORS & PACKAGING



MODEL	SIZE	COLORS
XS	140x95	  
S	160x95	  
M	180x95	  

PACKAGING	CONTENT	TOTAL UTS
Blister	10 uds	10
Master box	5 blisters of 10 uts	50



Airnatech Antiviral S.L.  
Ctra. Borriol Km 2,6. 12004. Castellón  
[info@ainartech.es](mailto:info@ainartech.es)  
[www.airnatech.es](http://www.airnatech.es)